
USS Sports Medicine\*  
**POLYSORB\* Meniscal  
Stapler XLS Template**

USS Sports Medicine\*  
**POLYSORB\* Meniscal Stapler  
XLS Template**

ENGLISH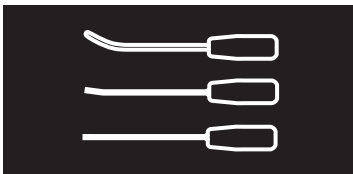

 **BEFORE USING PRODUCT,  
READ THE FOLLOWING INFORMATION THOROUGHLY.**

**IMPORTANT!**

This booklet is designed to assist in using the USS Sports Medicine\* POLYSORB\* Meniscal Stapler XLS Template. It is not a reference to surgical techniques.

**CAUTION:** Federal (USA) law restricts this device to sale and use by or on the order of, a physician.

**INDICATIONS**

The USS Sports Medicine\* POLYSORB\* Meniscal Stapler XLS Template is a reusable instrument used as a guide for selecting the appropriate shaped POLYSORB\* Meniscal Stapler XLS Single Use Loading Unit (SULLU).

**SCHEMATIC VIEW**

- A) 30 DEGREE L/R TEMPLATE (TOP VIEW / SIDE VIEW)
- B) UP-CURVE TEMPLATE (TOP VIEW / SIDE VIEW)
- C) STRAIGHT TEMPLATE (TOP VIEW / SIDE VIEW)

**POLYSORB\* MENISCAL STAPLER XLS TEMPLATE**

**REPROCESSING INSTRUCTIONS**

The POLYSORB\* Meniscal Stapler XLS Template has a stainless steel shaft with an anodized aluminum handle.

**Warnings:**

Detergents and solutions should have a pH between 7.0 and 9.5.

Sterilization temperature should not exceed 275° F (135° C).

The POLYSORB\* Meniscal Stapler XLS Templates are supplied non-sterile. Prior to use they must be cleaned and sterilized.

**Reprocessing Limitations:**

Repeated processing at the listed temperatures has minimal effect on the stainless steel instruments. Wear and damage due to use normally determine the end of instrument life.

**INSTRUCTIONS**

**Point of Use:**

Remove excess soil on reusable instruments with disposable wipes.

**Containment and Transportation:**

It is recommended that the reusable instruments be reprocessed as soon as possible following use. They should be covered with a moist towel if reprocessing cannot be performed immediately.

**Preparation for Cleaning:**

No disassembly required.

**Manual Cleaning:**

The reusable instruments should be cleaned thoroughly after every use to remove all traces of blood and debris. Pay particular attention to crevices while scrubbing instruments with a soft bristle brush in warm water (20 - 25° C) with a mild detergent/enzymatic cleaner (such as Alconox® or TERG-A-ZYME® solution)

**Automatic Cleaning:**

The reusable POLYSORB\* Meniscal Stapler XLS Template may be placed in an automatic dishwasher at 194° F (90° C).

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**Sterilization:**

The reusable POLYSORB® Meniscal Stapler XLS Templates are provided non-sterile and are for reuse. The POLYSORB® Meniscal Stapler XLS Templates may be steam sterilized by the hospital using the following process parameters:

**Prevacuum Steam Cycle**

Temperature: 270 - 275° F (132 - 135° C)

Time: 3 minutes minimum of exposure, wrapped

**Gravity Cycle**

Temperature: 250 - 255° F (121 - 124° C)

Time: 15 minutes minimum of exposure, wrapped

**Maintenance:**

Visually inspect the instruments for damage or wear prior to use. Repeated autoclaving may tarnish instrument surfaces.

**Packaging:**

Place items in a Polyethylene/Tyvek® pouch or standard hospital wrap.

**Storage:**

Store at room temperature.

**Additional Information:**

When sterilizing multiple instruments in one autoclave cycle ensure that the sterilizer manufacturer's stated maximum load is not exceeded.

**INSTRUCTIONS FOR USE**

The POLYSORB® Meniscal Stapler XLS Template is provided NONSTERILE and MUST be cleaned and sterilized prior to use.

- Reduce the meniscal tear so that the edges are coapted.
- Abrade the inner and outer edge of the tear site using the mechanical shaver and/or an arthroscopic rasp. Shaving or rasping the perimeniscal synovium, both superiorly and inferiorly, should also be performed to stimulate a vascular response. A standard posterior portal may also be used to facilitate rasping and preparation of meniscal and perimeniscal synovium.
- A cannula for introduction of the POLYSORB® Meniscal Stapler XLS Template is applied through the ipsilateral side of the tear for repairing posterior horn tears and through the contralateral side for repairing mid-third tears. Both sides are frequently used to optimally repair the full-length of the tear.
- Using the POLYSORB® Meniscal Stapler XLS Template, assess the tear shape and location. Use the template as a guide to determine which Meniscal Stapler SULU is appropriate to the pathology.

**CAUTIONS**

Only physicians having adequate training and familiarity with arthroscopic techniques should perform arthroscopic procedures. Prior to performance of any arthroscopic procedure consult the medical literature relative to techniques, complications, and hazards.

**CONTRAINDICATION**

The POLYSORB® Meniscal Stapler XLS Template is intended for use only as indicated.

**SERVICE AND REPAIR**

Contact United States Surgical, a division of Tyco Healthcare Group LP, at 1-800-423-8772 or 203-492-5550 for return goods authorization number. Instruments requiring service and repair should be returned to United States Surgical, Norwalk, CT 06856, USA. Unauthorized alterations, repair or modifications of these devices may jeopardize patient safety and will void all product warranties. Always include a repair purchase order number, a written description of the problem and a contact persons name, area code and phone number at your facility.

TEG-A-ZYME

**TERG-A-ZYME®** is a registered trademark of Alconox, Inc. Alconox® is a registered trademark of Alconox, Inc. Tyvek® is a registered trademark of Dupont.

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TEG-A-ZYME® is a registered trademark of Alconox, Inc. Alconox® is a registered trademark of Alconox, Inc. Tyvek® is a registered trademark of Dupont.

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TEG-A-ZYME® is a registered trademark of Alconox, Inc. Alconox® es marca registrada de Alconox, Inc. Tyvek® es marca registrada de Dupont.