USS Sports Medicine*
ArthroSew*
Single Use Automated Suturing Device with
8.6 mm ArthroPort* Cannula Set
ENGLISH

BEFORE USING PRODUCT, READ THE FOLLOWING INFORMATION THOROUGHLY.

IMPORTANT!
This booklet is designed to assist in using this product. It is not a reference to surgical techniques.

CAUTION: Federal (USA) law restricts this device to sale, and use by or on the order of, a physician.

INDICATIONS
The USS Sports Medicine* ArthroSew* suturing device has indications in arthroscopic surgery for the placement of sutures through soft tissue and/or connective tissue.

The USS Sports Medicine* 8.6 mm ArthroPort* cannula set contains a blunt obturator and an 8.6 mm threaded cannula. The USS Sports Medicine* 8.6 mm cannula set has application in arthroscopic procedures to establish a port of entry for instrumentation.

EFFECTS
The USS Sports Medicine* ArthroSew* suturing device has two jaws. A sutured needle, loaded from a single suture unit, is held in one jaw and can be passed to the other jaw by closing the handles and flipping the toggle levers. The USS Sports Medicine* ArthroSew* device is designed for introduction and use through all cannulas 8.4 mm or larger.

SCHEMATIC VIEW
A) METAL BARS
B) JAWS
C) SHAFT
D) TOGGLE LEVERS
E) HANDLES
F) ARROWS
G) BLACK RELOADING BUTTON
H) RED INDICATOR BOXES
I) THREADED CANNULA
J) BLUNT OBTURATOR

LOADING

- Prior to loading, ensure that the metal bars are extended. If the metal bars are not extended, follow the unloading instructions prior to loading the device.
- Insert open jaws of the device, with the handle logo facing up, into the single suture unit. Press firmly on the apex of the jaws to ensure that the jaws are fully seated.
- To activate the toggle levers, completely squeeze the handles and pull both toggle levers back until you hear an audible click.

NOTE: The black reloading button will cover the red indicator boxes.

F) ARROWS
D) TOGGLE LEVERS
H) RED INDICATOR BOXES

- Press down on the single suture unit tab. Remove the device and entire length of suture.

NOTE: Test to ensure that the needle is fully seated by passing the needle back and forth between the jaws of the device.

INSTRUCTIONS FOR USE
THE PRESENCE OF THE BLUNT TIP ON THE DISPOSABLE OBTURATOR IS NOT A SUBSTITUTE FOR ADHERENCE BY THE USER TO PROPER ARTHROSCOPIC TECHNIQUES.

- Incise the selected area adequately to accommodate the entire diameter of the blunt obturator. A Kelly clamp or scissors can be used to aid in dissection and spreading of tissue to accommodate the introduction of the cannula set.
- Introduce the cannula set through the incision by turning clockwise in a continuous downward motion.
- Remove the obturator, leaving the cannula in place. Instruments of appropriate size can be easily introduced and removed through the threaded cannula as needed.
- Prior to introduction of the ArthroSew* suturing device through a cannula, ensure that the jaws are locked in the closed position. To lock the jaws in a closed position, squeeze the handles and push either of the toggle levers until the toggles are aligned with the arrows on the body housing.
- The ArthroSew* suturing device is designed for introduction and use through all appropriately sized cannulas 8.4 mm or larger.
- Once the device is inside the joint, the jaws are opened by squeezing the handles and pushing either toggle lever completely forward. Once the handles are released, the needle is exposed and ready for use.

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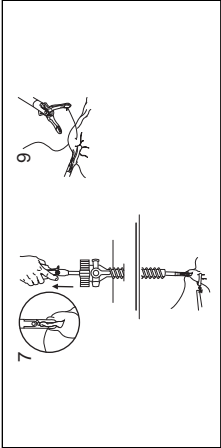
7. To guide the needle through the tissue, close the jaws by completely squeezing the handles and reversing the position of the toggle levers until they stop.

NOTE: toggles may only be activated when jaws are completely closed.

Failure to do so may damage the instrument.

8. The needle has now been passed to the opposite jaw of the ArthroSew® device.

9. Open the jaws by releasing the handles and pull the suture through the tissue. At this point a simple or mattress sitch can be utilized.

10. The arthroscopic knot by advancing knot to the soft tissue with a knot pusher. Repeat procedures as necessary

NOTE: Prior to removal of the device, the jaws must be in a closed position.
11. Upon completion of the procedure, the cannula is removed by turning counterclockwise

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All Rights Reserved. T2402 Made in USA. 04
European Representative: Tyco Healthcare UK Ltd., Gosport, PO13 0AS, UK